ELECTRIC BREST PUMP

HANDS

FREE

lorelli[®]
designed in EU

V 1.1

**MANUAL
INSTRUCTION**

find us on
www.lorelli.eu

CONTENT

EN	Manual Instruction.....	7
BG	Инструкция за употреба.....	13
RO	Instrucțiuni de utilizare.....	19
FR	Mode d'emploi.....	25
IT	Istruzione per l'uso.....	31
GR	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ ΚΑΡΕΚΛΑΣ.....	37
ESP	Instrucciones de uso.....	43
RU	Инструкция по эксплуатации.....	49
SRB HR	Uputstvo za upotrebu.....	55
ME BIH		
HU	Használati utasítás.....	61
AL	Insruksion për përdorim.....	67
NL	Gebruikshandleiding.....	73
DE	Gebrauchsanweisung.....	79
PL	Instrukcja użytkownika.....	85
MK	Упатствата за употреба.....	91
AR	دليل التعليمات.....	97
CZ	Návod k použití.....	103
TR	Kullanım talimatı.....	109

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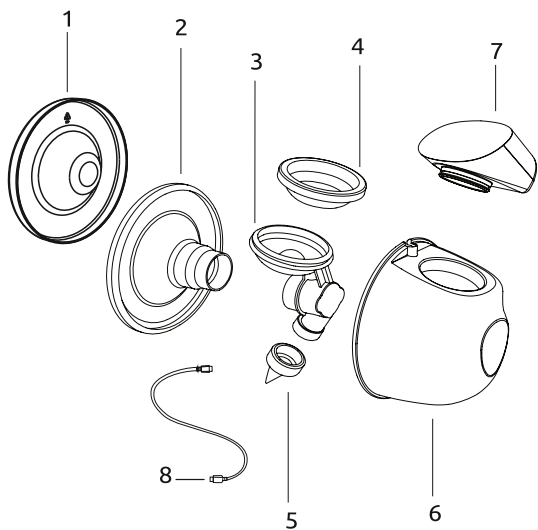
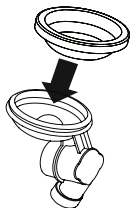
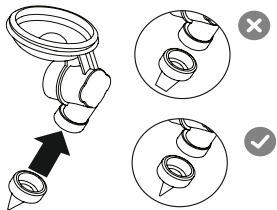
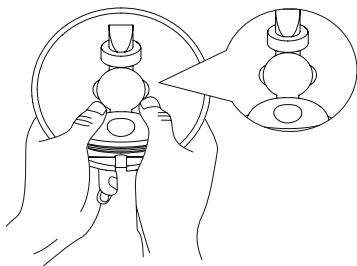
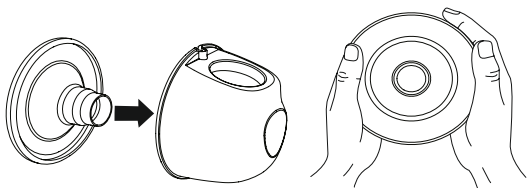
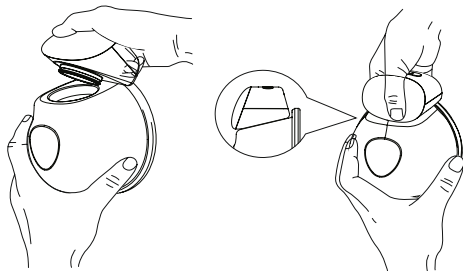
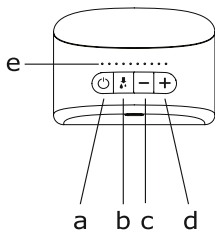
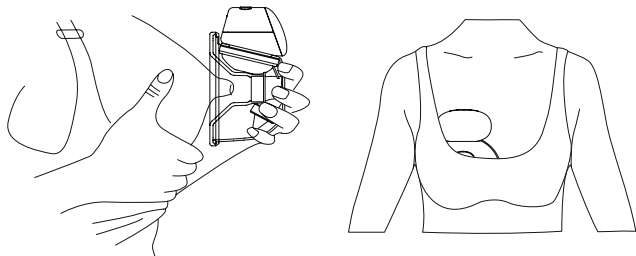
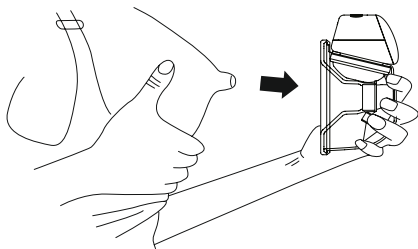
✗ WON'T WORK

Laying back (or side)

Before using the breast pump carefully read this user manual and keep it safe for future reference.

EN

Important safety information.

Before using the breast pump carefully read this user manual and keep it safe for future reference.

Intended use

The electric breast pump is intended for expressing and collecting breast milk from the breasts of nursing women. The device is intended for one user

SAFETY TIPS

Contraindications

WARNING! Never use the breast pump while pregnant, as pumping can induce labour.

To prevent injury and damage, important notes are explained using different symbols. The symbols and their meanings are as follows:

WARNING! – indicates potentially hazardous situations which, if not avoided, could result in death or serious injury.

ATTENTION! - indicates potentially hazardous situations which, if not avoided, may result in minor or moderate injuries.

WARNING

Warnings to avoid choking, strangulation and injury:

- Do not allow children or pets to play with the motor unit, adapter or accessories.
- Children should be supervised to ensure that they do not play with the appliance
- **Always unplug** the breast pump immediately after use.
- Do not leave the breast pump unattended while plugged in.
- This breast pump is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities or lack of experience and knowledge. Such persons can only use this breast pump if they are supervised or have been instructed by a person responsible for their safety on how to use the device.

Warnings to avoid electric shock and burns:

- Inspect the breast pump, including the adapter, for signs of damage before each use. Do not use the breast pump if the adapter or plug has been damaged, if the device is not working properly or if it has been dropped or submerged in water.
- Use only the suggested adapter and specified voltage to prevent adapter overheating, damage or accidents.
- Do not touch the AC adapter or pump during a thunderstorm.
- Do not place the pump/battery/AC adapter in direct sunlight or rain, near heaters or other hot appliances, or in places exposed to extremely high or low temperatures, high humidity or dust.
- Do not place heavy objects on the pump or damage the cable or plug. Do not stretch or twist the cable. Always hold the AC adapter when unplugging. Make sure the plug is firmly inserted into the outlet. Do not use an outlet that is loose.

- Do not allow the pump/battery/AC adapter to get wet.
- Do not disassemble or modify the pump/battery/AC adapter. Do not throw the battery into fire or expose it to heat.
- Immediately unplug the AC adapter and remove the battery and stop using the pump if you notice an abnormal odour, heat, discolouration, deformation, or anything unusual while using or storing the pump.

Warnings to avoid poisoning, infection and to ensure hygiene:

- For sanitary reasons, the breast milk pump is intended for repeated use by one user only.
- Before each use clean, rinse and disinfect all parts of the pump except the drive unit and the adapter.
- Only store breast milk that has been expressed with a cleaned and disinfected pump.
- Do not use the breast pump if the silicone diaphragm appears damaged or defective.
- Do not clean the pump parts with antibacterial or abrasive cleaning agents.

Precautions to avoid problems and pain in breasts and nipples:

- Do not attempt to remove the pump while it is attached to the breast with suction. Turn off the device and break the seal by sliding a finger between your breast and the pump funnel. Then remove the pump from the breast.
- Never use the breast pump when you are drowsy or sleepy to avoid inattention during use.
- Always turn off the breast pump before removing it from the breast to release the seal.
- Never use accessories or parts from other manufacturers or those not specifically recommended by the manufacturer. This could cause the device to malfunction and may affect electromagnetic compatibility (EMC). Using such accessories or parts will void your warranty.
- Do not pump for more than 5 minutes without stopping if no breast milk comes out. Try pumping at a different time of day.
- If the process becomes very uncomfortable or painful, stop using the pump and consult your lactation consultant.
- If the pressure causes you discomfort or pain, turn off the device, break the seal by sliding a finger between the breast and the pump body, and remove the pump from your breast.
- Never drop or insert any foreign objects into the openings.

ATTENTION

Warnings to prevent personal injury, product damage and malfunction:

- Do not allow the adapter and motor unit to come into contact with water.
- Keep the adapter and motor unit away from hot surfaces to avoid overheating and deformation of these parts.
- Never place the motor unit or adapter in water, in a dishwasher or in a sanitiser.
- Although the breast pump complies with the applicable EMC directives, it may still be sensitive to stronger emissions and/or cause interference with other equipment. This may cause the breast pump to switch off or to go into error mode. To prevent interference, keep other electrical equipment away from the breast pump during expression (see 'Technical information').
- There are no user-serviceable parts in the motor unit of the breast pump. No equipment modification is permitted. Doing so will void your warranty.
- Do not drop or hit the pump/AC adapter. Do not press hard on the LCD display.

— The drive unit and adapter do not require cleaning and sterilization as they do not come into contact with breast milk.

— Never use alcohol or other organic solvents to clean the pump.

— Do not use the pump in a way or for purposes other than those described in the instructions. Doing so may result in accidents or damage to the pump.

Picture 1:

1. Safety cap	- 1 pc.	6. Container	- 1 pc.
2. Silicone massage cushion	- 1 pc.	7. Pump	- 1 pc.
3. Valve base -	- 1 pc.	8. USB power cable	- 1 pc.
4. Silicone membrane -	- 1 pc.	9. Adapter	- 1 pc.
5. Valve	- 2 pcs.		

PARTS

Cleaning and disinfection

PREPARING FOR USE

The drive unit and adapter do not require cleaning and disinfection as they do not come into contact with breast milk. Before the first use, clean and disinfect all other parts of the breast pump. Clean these parts after each subsequent use and disinfect them before each subsequent use. Dry well. **Attention!:**

Please dry the silicone membrane before use to avoid damage to the motor.

Attention!: Never immerse the drive unit or adapter in water, in a dishwasher or in a sanitiser because this will cause irreversible damage!

1. Disassemble the pump completely. Remove the valve as well. Be careful when removing the valve and when cleaning it. If it breaks, your breast pump will not work properly.

2. Clean all parts except the drive unit and adapter in hot water with a little mild liquid dish soap and then rinse thoroughly. To clean the valve, rub it gently between your fingers in warm water with a little liquid detergent. Do not insert objects into the valve as this may damage it.

Warning! - Do not use antibacterial or abrasive cleaning agents.

Note: Make sure you clean and disinfect the relevant parts of the breast pump. Wash your hands thoroughly before handling the cleaned parts!

Warning! - Be careful as the cleaned parts may still be hot. Start assembling the breast pump only when the cleaned parts have cooled.

Buttons (Picture 5)

USAGE

A. ON/OFF button; B. Mode button; C. Suction decrease; D. Suction increase; E. - Suction strength indicator;

Red light - Massage mode;

Blue light - Pumping mode;

Purple light - Combined mode;

Battery indication when connected to the charging adapter:

Green - Battery fully charged (constantly lit);

Orange - Average battery charge;

Red - Low battery;

Red - Flashing signal - minimum battery charge;

BATTERY OPERATION

ATTENTION!!! Before first use, fully charge the battery for 8 hours continuously!

Connect the USB plug of the power cable to the socket of the pump, and the other side of the power cable to the adapter, which is not included in the set of the device.

The battery is fully charged when the indicator turns constant green.

Important! The battery cannot be charged for more than 24 hours.

PUMP ASSEMBLY

1. Press the silicone membrane into the valve base from above. Secure it tightly around the edge by pressing down with your fingers to ensure a perfect seal. (Picture 2)
2. Insert the valve into the valve base from underneath. Push the valve in as far as possible so that it covers the edge of the base. (Picture 2)
3. Press the assembled valve base against the container so that the silicone membrane lines up with the opening. Ensure that the flap membrane is firmly attached to the container edge. (Picture 3)
4. Place the silicone safety cushion, pre-centring and pressing against the container, so that the centre of the cushion covers the opening of the valve and the periphery seals well the edge of the container. (Picture 4)
5. **Place the pump in the opening of the container and press. Make sure the pump is properly attached to the container!** (Picture 5)

SETTINGS

 ON/OFF button - massage mode starts automatically for 2 minutes, then changes to pumping mode. By default, the pumping time is 30 minutes, the pump will automatically turn off after the time is up.

 Mode changing button - after successively pressing the indicated button, the modes start to change successively to massage, pumping and combined.

When the pump is first turned on, the general functions are automatically set to silent mode (weak mode), which automatically increases the pump modes, sound and light indicators by double pressing the on/off button. Apart from that, the pump also offers a manual increase or decrease of each mode in the order of 10 degrees (reflected with a light indication), using the + and - buttons.

Important: The level of milk in the bottle must not exceed the level of the valve!

Don't be concerned if the flow of milk suddenly stops. Continue pumping calmly, but not for more than 5 minutes. If you are not expressing more milk, try another time of day. It is not necessary to use all modes of operation, only the one that is most comfortable for you. The pump automatically switches off after 30 min.

WHEN TO EXPRESS BREASTMILK

If breastfeeding goes well, it is advisable (unless advised otherwise by your healthcare professional/breastfeeding consultant) to wait until your milk supply and breastfeeding schedule are established (normally at least 2 to 4 weeks postpartum) before you start expressing.

Exceptions:

- If you have trouble breastfeeding at first, regular pumping can help build and maintain your milk supply.
- If you are expressing breast milk for your baby to be given in the hospital.
- If your breasts are engorged (painful or swollen), you can express some breast milk before or between feeds to ease the pain and help your baby latch more easily.
- If you have sore or cracked nipples, you may wish to express your milk until they are healed.
- If you are separated from your baby and wish to continue to breastfeed when you are reunited, you should express your milk regularly to stimulate your milk supply.

You need to find the optimum times of the day to express your milk, for example just before or after your baby's first feed in the morning when your breasts are full, or after a feed if your baby has not emptied both breasts. If you have returned to work, you may need to express during a break. Using a breast pump requires practice and it may take several attempts before you succeed. Fortunately, the Lorelli electric breast pump

is easy to assemble and use so you will soon get used to expressing with it.

OPERATING THE BREAST PUMP

1. Wash your hands thoroughly and make sure your breasts are clean. Take one of the allowed positions shown in pictures 9 and 10. **Attention, picture 11 shows the unfavourable positions in which the pump does not function properly and there is a possibility of damage.**

2. Press the assembled pump body against the breast. Make sure your nipple is centred so the massage cushion can create an airtight seal. Make sure your nipple fits properly in the pad.

3. Press the on/off button. The breast pump automatically starts working in massage (stimulation) mode and you can change the massage level using the increase and decrease buttons.

4. When the breast milk flows, you can change the mode from massage to normal pumping - via the mode button and choose the most comfortable level.

ATTENTION: Don't always use the maximum pumping level, choose the one most comfortable for you.

Note: Don't worry if your milk doesn't flow right away. Keep calm and keep pumping. The first few times you use the breast pump, you may need to set it to a stronger suction setting to get your breast milk flowing.

5. Depending on your personal comfort, you may decide to use a stronger suction mode. You can always switch back to a lower suction mode.

6. When you are done expressing, turn off the pump with the ON/OFF button and carefully remove the pump body from your breast.

7. Separate the bowl from the motor unit, observing the appropriate sanitary measures. Use the breast milk as intended (feed the baby or store the milk).

8. Remove the motor unit and silicone membrane.

9. Clean the rest of the used parts of the breast pump according to the instructions in the "Cleaning and disinfection" section.

STORING BREAST MILK

Warning: To ensure proper hygiene, only store breast milk that has been expressed with a cleaned and disinfected pump.

- Breast milk can be stored in the refrigerator (not on the door) for up to 48 hours. Expressed breast milk should be refrigerated immediately. If you store breast milk in the refrigerator and add to it during the day, only add breast milk that has been expressed in a disinfected bottle.

- Breast milk can be stored in the freezer for up to three months if it is in disinfected bottles with a disinfected screw ring and sealing disc. Place a label on the bottle marking clearly the date and time of expression and use the older breastmilk first.

- If you are going to feed your baby the expressed breast milk within 48 hours, you can store it in the refrigerator in an assembled bottle.

WARNING!

Always put expressed breast milk in the refrigerator or freezer immediately.

Only store breast milk that has been expressed with a disinfected breast pump in disinfected bottles.

Never refreeze thawed breast milk.

Never add fresh breast milk to already frozen breast milk.

TROUBLESHOOTING

Problem: I experience pain when I use the breast pump.

Solution: One possible cause is excessive suction. You don't need the full force of suction produced by the pump. Initially, use "Massage" mode for massage stimulation, and then you can switch to pumping mode. If the pain does not subside, stop using the breast pump and consult your lactation consultant. If the pump does not come off the breast easily, you can break the seal by sliding your finger between the breast and the massage cushion.

Problem: The breast pump is not working.

Solution: First, check that you have assembled the breast pump correctly. To prevent interference, keep other electrical equipment, such as mobile phones or laptops, away from the breast pump during expression. Check that the battery is charged.

Problem: I don't feel any suction.

Solution: Check that you have correctly assembled the breast pump by following the steps in the "Preparation for Use" section. Make sure the valves are properly placed and check for damage or wear. Replace the valve if necessary. Also, make sure the pump is placed correctly against the breast to create a vacuum.

Problem: The pump has too much suction.

Solution: Follow the instructions below if you experience too much suction from the breast pump:

Make sure you only use parts recommended by the manufacturer.

Make sure you have assembled the breast pump with the massage cushion: not using the massage cushion may cause too much suction.

When you use the breast pump for the first time, you may feel too strong suction at first. Practising can help. If the problem persists, consult your lactation consultant.

USAGE AND STORAGE CONDITIONS

Keep the breast pump out of direct sunlight as prolonged exposure to the sun can cause discoloration.

Store the breast pump and its accessories in a safe, clean and dry place.

If the appliance has been stored in a hot or cold environment, place it in the usage environment to let it reach a temperature within the usage conditions

(10°C to 30°C) before you use it.

TECHNICAL SPECIFICATIONS

1. Adapter input: AC 100–240V; 50/60Hz 0.2Amax;
2. Adapter output: DC 5.0V, 5W, 1.0A
3. Rated voltage: 5.0V
4. Rated current: 1000mAh
5. Lithium battery 3.7V 1100mAh
6. Fixed time : 30 minutes
7. Suction power: 60mmhg--150mmhg
8. Noise level: <60dB
9. Environmental conditions for operation : Temperature 10-30 °C, Relative humidity 15%-93%, Atmospheric pressure 700-1060hpa
10. Environmental conditions for storage : Temperature -25-50 °C, Relative humidity 15%-93%, Atmospheric pressure 700-1060hPa
11. IP Class: IPX0 .

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NINGBO YOUHE ELECTRICAL APPLIANCE TECHNOLOGY CO., LTD
(NINGBO YOUHE MOTHER AND BABY PRODUCTS CO., LTD)
567# Xintang Road, Fuhai, Cixi, Zhejiang, China

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宁波优合母婴用品有限公司
NINGBO YOUHE MOTHER & BABY PRODUCTS CO., LTD

DECLARATION OF CONFORMITY

Name and address of the manufacturer:

NINGBO YOUHE ELECTRICAL APPLIANCE TECHNOLOGY CO., LTD
(NINGBO YOUHE MOTHER AND BABY PRODUCTS CO.,LTD)
567# Xintang Road, Fuhai Cixi, Zhejiang, China

We hereby declare that our product, item number P1 (THE INS) and the components of our product delivered to DIDIS Ltd as 1022070 ELECTRIC BREAST PUMP HANDS FREE comply with the requirements of the below listed directive and European standards:

1. DIRECTIVE 2014/30/EU OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 26 February 2014 (EMC)

The product conforms to the following standards:

EN IEC 55014-1

EN IEC 61000-3-2

EN 61000-3-3

EN IEC 55014-2

2. DIRECTIVE 2014/35/EU OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 26 February 2014 (LVD)

The product conforms to the following standards:

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MOTHER&BABY PRODUCTS CO.,LTD.
Importator:Didis Ltd.
PRODUCTS CO.,LTD. Zhejiang, China